

Quick Start Guide

Pairing Pure Charge&Go BCT IX to an Android phone



Your hearing aids use two types of Bluetooth® connections.

- Bluetooth® LE is used to connect each hearing aid individually to the Signia app for remote control adjustments. Pairing this connection is done through the Signia app.
- Bluetooth® Classic is used to connect your hearing aids jointly for streaming calls and audio. Pairing this connection is done through your phone's Bluetooth menu.

Follow the simple steps below to pair both types of Bluetooth connections. *These instructions are intended as general guidance, as menu details may vary between different Android phones.*

Step 1

Download the Signia app (version 2.7.0 or later).

Step 2


Ensure your phone's Bluetooth is switched on.



Note: If your hearing aids are already paired to another device nearby, turn off the Bluetooth® on that other device during the pairing process.


Step 3

Open the **Signia app**.

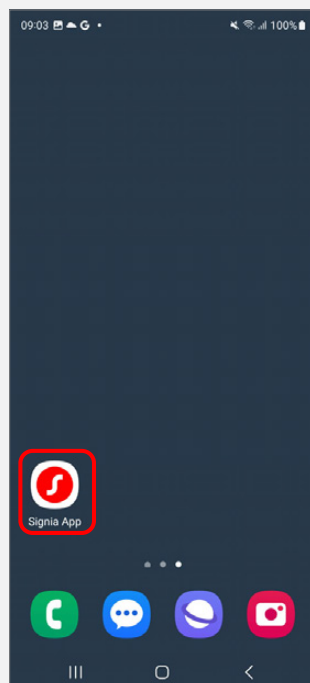


Step 4

Tap the Bluetooth pairing option.

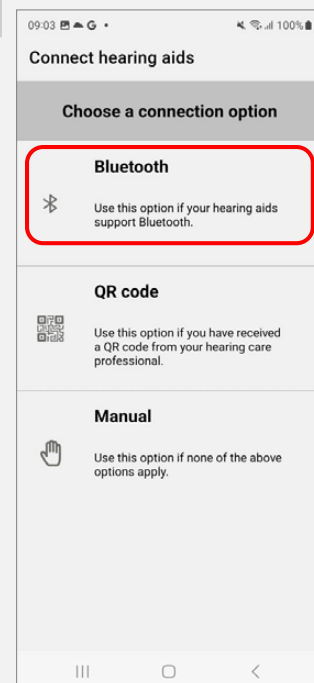


Step 5

Turn your hearing aids off and back on – this will put them in pairing mode for approximately 3 minutes. Tap **OK**.

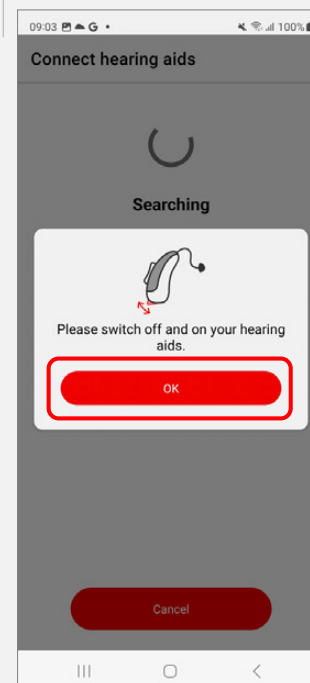



Step 6

When your hearing aids appear, **tap their name** to select.

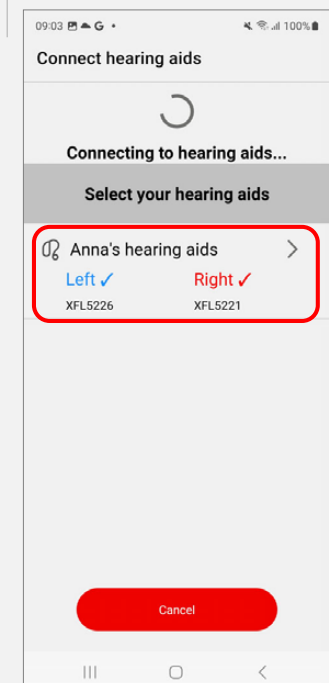

Step 7


A Bluetooth pairing request will appear - **tap Pair**. If you are pairing two hearing aids, a second request will appear a few seconds later - **tap Pair** again.

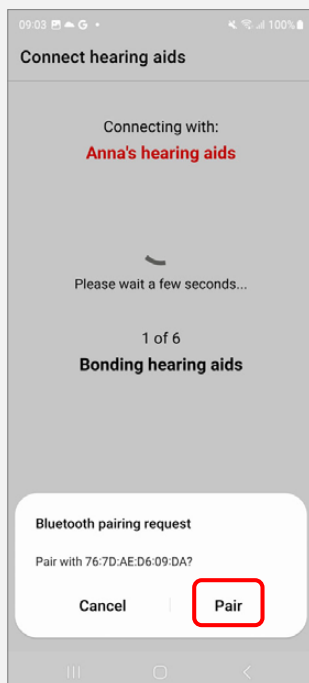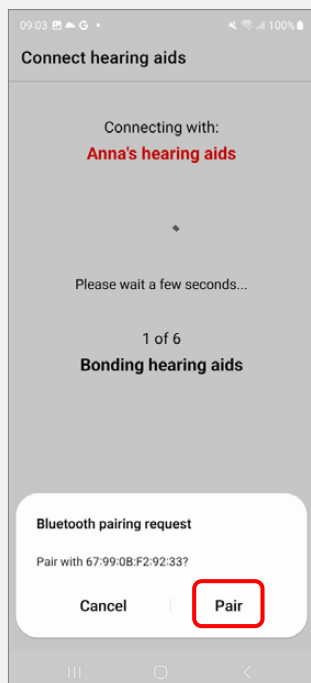

Step 8


The hearing aids are now paired to the Signia app.

Tap the arrow next to Activate streaming to pair them for audio streaming.

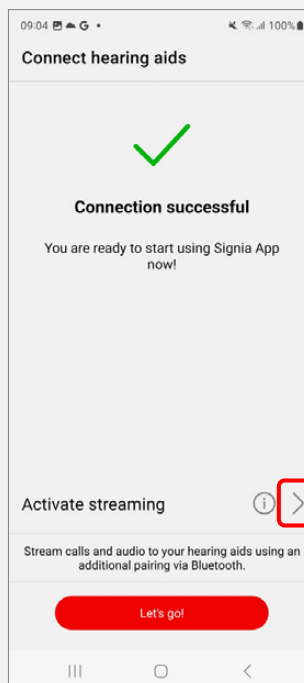


Step 9

Tap Open smartphone settings to directly open your phone's Settings menu.

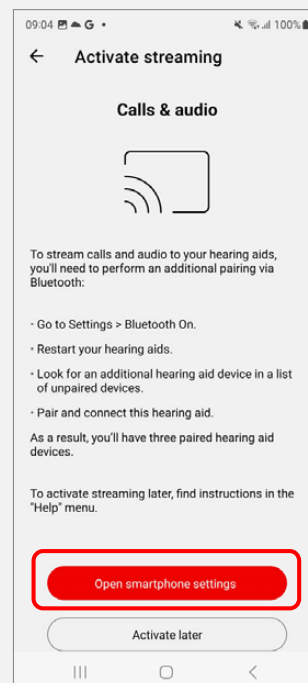

Step 10



Tap Bluetooth to open the Bluetooth menu.

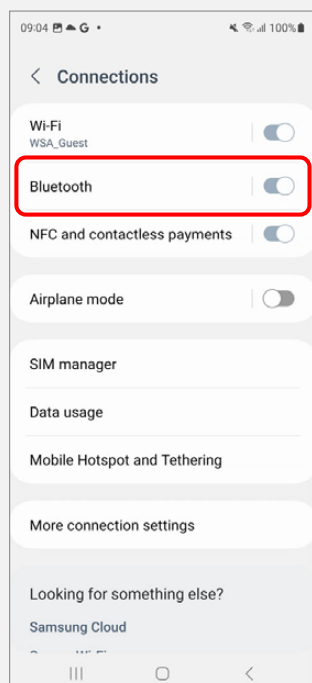

Step 11

Your hearing aids will be listed under Paired Devices, with names beginning LE_ (you may see one or two listings depending on phone model).

They will *also* be listed once under Available Devices. **Tap this available hearing aid**, then **tap Pair** when prompted – **this will pair your hearing aids jointly for audio streaming**.

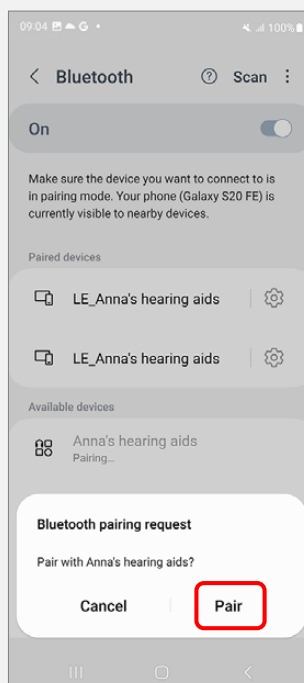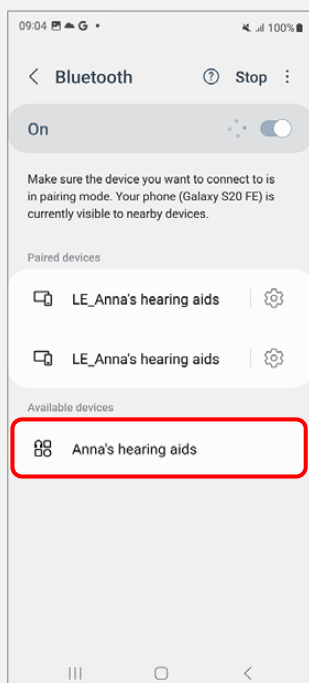

Step 12

Ensure your hearing aids are in the correct wearing position.

You are now ready to enjoy streaming audio to your hearing aids.

