

Quick Start Guide

Pairing Pure Charge&Go BCT IX to an iPhone



Your hearing aids use two types of Bluetooth® connections.

- Bluetooth® LE is used to connect each hearing aid individually to the Signia app for remote control adjustments. Pairing this connection is done through the Signia app.
- Bluetooth® Classic is used to connect your hearing aids jointly for streaming calls and audio. Pairing this connection is done through your phone's Bluetooth menu.

Follow the simple steps below to pair both types of Bluetooth connections.

Step 1

Download the Signia app (version 2.7.0 or later).



Step 2


Ensure your phone's Bluetooth is switched on.



Note: If your hearing aids are already paired to another device nearby, turn off the Bluetooth on that other device during the pairing process.


Step 3

Open the Signia app.

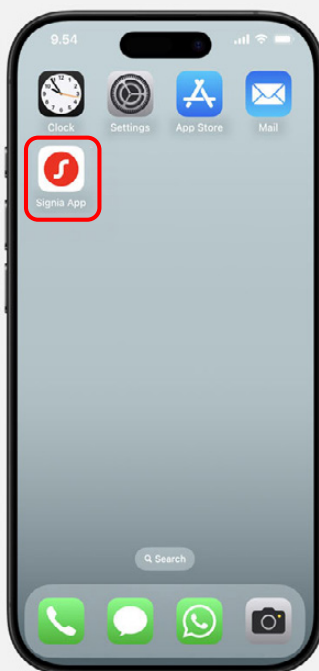


Step 4

Tap the Bluetooth pairing option.

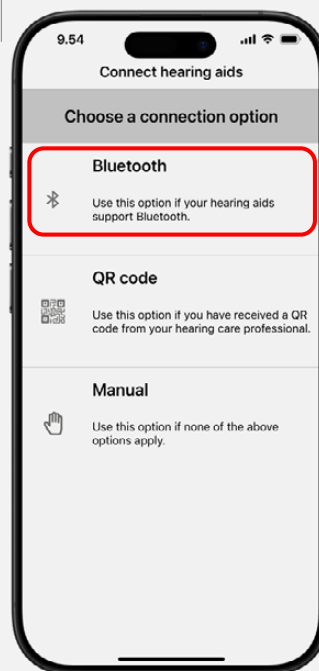


Step 5

Turn your hearing aids off and back on to put them in pairing mode for approx. 3 minutes. Tap OK.

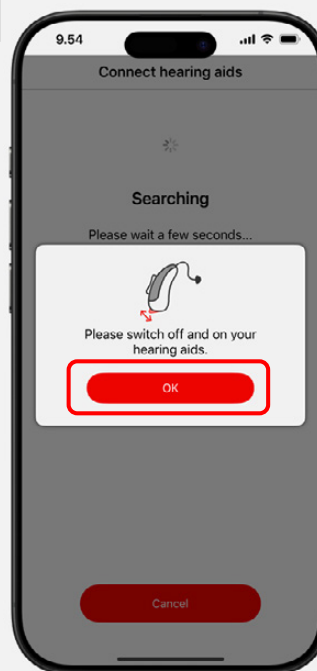


Step 6

When your hearing aids appear, tap their name to select.

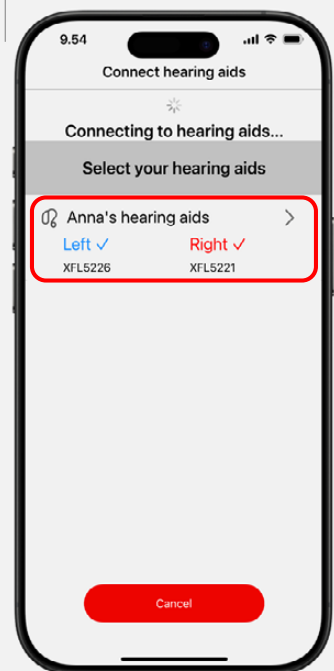

Step 7

A Bluetooth pairing request will appear **tap Pair**. If you are pairing two hearing aids, this will happen twice.

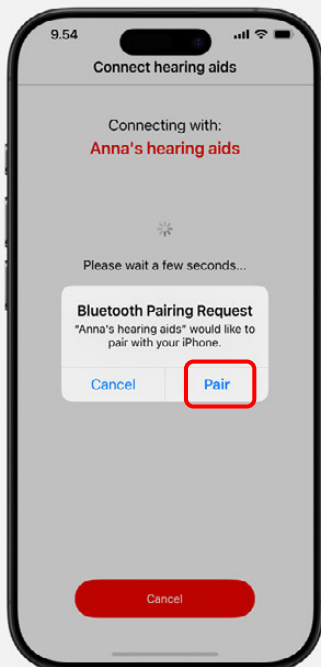


Step 8

The hearing aids are now paired to the Signia app.
The following steps will pair them for audio streaming.



Step 9

Open your phone's **Settings** menu, then **tap Bluetooth** to open the Bluetooth menu.
Turn your hearing aids off and back on to put them in pairing mode.

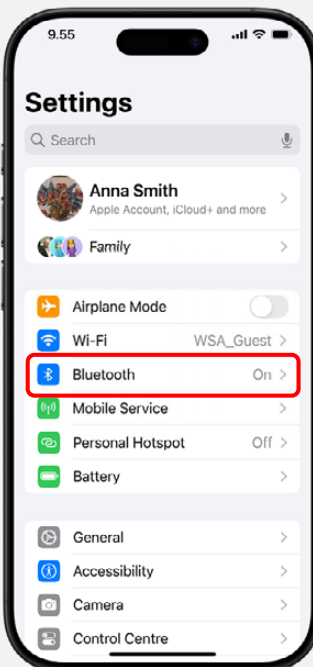


Step 10

You will see two paired hearing aids under My Devices with names beginning **LE_**.
These are the pairings that are connected to the Signia app.




Step 11

A third hearing aid listing will appear under **Other Devices** – **tap to pair it**.
This is the pairing for the **Bluetooth streaming connection**.

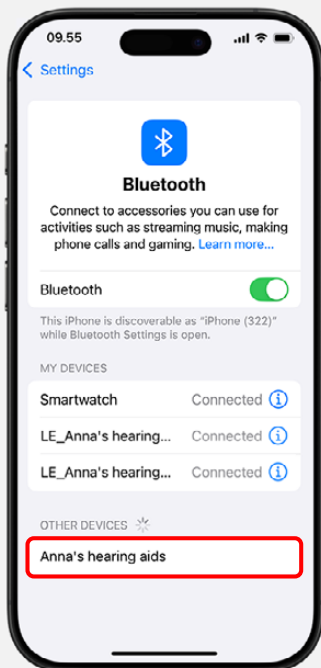


Step 12

To optimize audio streaming settings, **tap the ⓘ icon** next to the hearing aid name that was just paired for Bluetooth streaming.

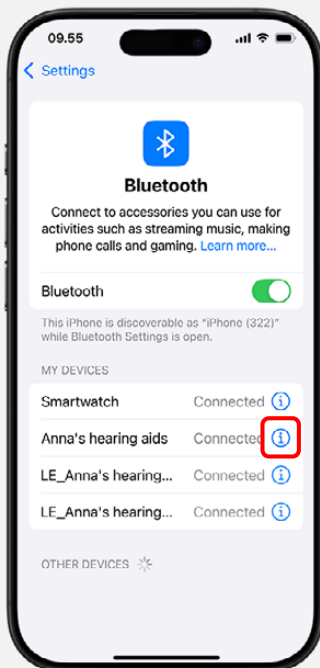

Step 13

Tap the arrow to the right of **Device Type** and **select Hearing Aid** from the list of options that will open.

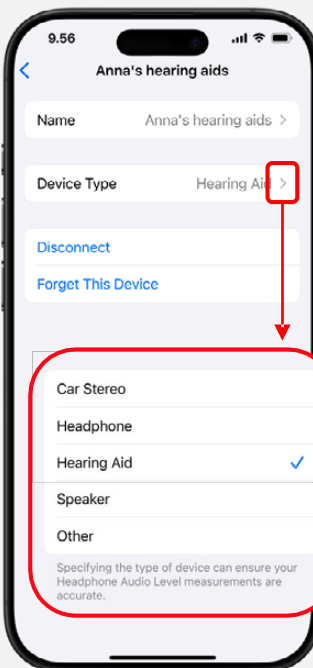

Step 14

Ensure your hearing aids are in the correct wearing position.
You are now ready to enjoy streaming audio to your hearing aids.

